DE Montageanleitung
GB Assembly Instructions
FR Instructions de montage
ES Instrucciones de montaje
IT Istruzioni per il montaggio
NL Montage-instructie

PT Instruções de montagem
DK Montagevejledning
SE Monteringsanvisning
PL Instrukcja montażu
GR Οδηγία μοντάζ
CZ Návod k montáži

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Tel: +49 2103 573-0 · Fax: +49 2103 573-190

service@wenko.de · www.wenko.com

GB Bathroom stool, round, with swivel function,
Secura Premium

Model: 23742*

CAUTION!

Please make sure that all legs have the same height setting before using the seat/stool. All locking buttons must be engaged before subjecting to loads.

1. Areas of use:

This product may only be used in private households! Not suitable for use in rehabilitation facilities and for the medical care of persons. This product is intended for use in the wet area of the shower.

max. load capacity 130 kg.

Recommended service life: 15 years from production date

2. Safety instructions

Pay attention to a firm hold! Smooth or wet surfaces can worsen the non-slip properties. Check connecting elements regularly. The product should be checked regularly for stability and damage, and reuse is only permitted after any necessary repairs have been professionally carried out. Please keep these instructions, as they contain important information. Do not step on the seat.

Do not use to lift persons!

Accidents resulting in serious injuries may occur if the handles are used for incorrect purposes e.g. as gymnastic equipment or steps. Wenko Wenselaar GmbH & Co KG shall not assume any liability whatsoever for incorrect use!

3. Care instructions:

Clean with a damp cloth.

Do not use any sandy abrasive agents or scrub sponges for cleaning.

4. Exclusions:

Wenko does not assume any liability for damage to persons and property caused by improper mounting, incorrect use of the handles, in particular by non-observance of the warning and safety instructions listed above.

For after-sales service please contact the manufacturer directly:

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Wenko-Wenselaar GmbH & Co. KG